


## When Managing Bioburden is a Priority

- A unique antibacterial wound dressing that can be used under Ostomy barriers
- Manages bioburden, which helps the body's own immune system restore bacterial balance
- Broad spectrum activity against a wide variety of bacteria and yeast commonly found in wounds
- No hydration required

Hydrofera**BLUE**®  
■■■■

## Site Preparation

- Prior to the initial preparation of the Hydrofera Blue® Ostomy dressing, the wound should be debrided of any necrotic tissue
- The wound should be cleansed with normal saline or an appropriate cleansing solution

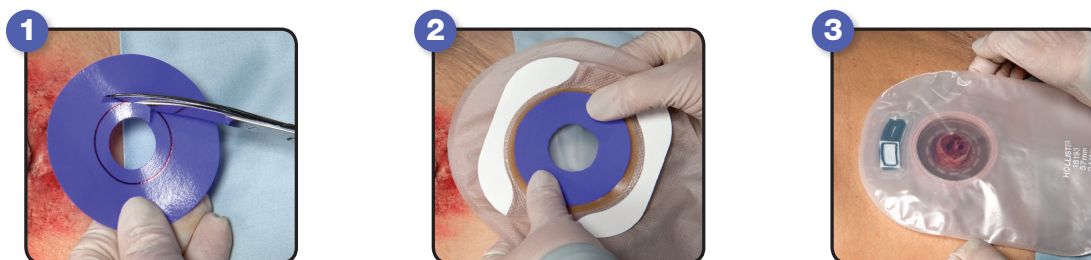

## Application Instructions

- 1 Trim the inner diameter of the Hydrofera Blue® Ostomy dressing to fit around the stoma. If needed, trim the outer edge of the dressing as well, so there is at least 1/2" of the skin barrier extending beyond the outer edge of the dressing.
- 2 Apply the film side of the dressing to the Ostomy skin barrier.
- 3 Apply the skin barrier to the skin.

A new Hydrofera Blue® Ostomy dressing should be applied when the skin barrier is changed. Hydrofera Blue® dressings should be changed every 1-3 days depending on the level of exudate and the condition of the wound.

### For extra security holding the skin barrier in place:

- Consider using an Ostomy belt or skin barrier with tape border
- Consider using a larger skin barrier to ensure coverage over the entire wound area, and so there is adequate skin barrier around the outer circumference of the Hydrofera Blue® Ostomy dressing

## Hydrofera Blue® Ostomy Dressing

Product No	Product Size	Quantity/Box	HCPCS CODES*
HBRF2650	2.5" (6.4 cm) Diameter Ostomy Dressing	10	A6209

### Hydrofera Blue CLASSIC® dressings also available:

Product No	Product Size	Quantity/Box	HCPCS CODES*
<b>Standard Dressings</b>			
HB2214	2" x 2" (5 cm x 5 cm)	10	A6209
HB4414	4" x 4" (10.2 cm x 10.2 cm)	10	A6209
HB4514	4" x 5" (10.2 cm x 12.7 cm)	10	A6210
HB6614	6" x 6" (15.2 cm x 15.2 cm)	10	A6210
HBTO906	9-mm Tunneling Dressing (1.2 g)	10 pair	A6215
<b>Heavy Drainage Dressings</b>			
HBHD4450	4" x 4" x 0.5" thick (10.2 cm x 10.2 cm x 1.2 cm)	5	A6209
HBHD6675	6" x 6" x 0.75" thick (15.2 cm x 15.2 cm x 1.9 cm)	5	A6210

The BSN Medical / Essity Inc. Reimbursement Disclaimer:

The Centers for Medicare & Medicaid Services (CMS) uses Palmetto GBA LLC., as the Pricing, Data Analysis, and Coding (PDAC) Contractor. The PDAC Contractor maintains the Product Classification List, located at <https://www4.palmettogba.com/pdac.dmecs/>. The Product Classification List is a searchable database containing products that have received HCPCS coding verification from PDAC and their corresponding HCPCS codes. BSN Medical Inc., however, makes no representations as to the accuracy of the information contained within the PDAC database, nor any representations as to whether its products are reimbursable under any government sponsored healthcare program and/or private-insurance program. It is the provider's sole responsibility, in consultation with the insurer, if necessary, to determine medical necessity, ensure coverage criteria is met, and submit appropriate codes, modifiers, and charges for services rendered. RV: 12/24



Hydrofera®,  
an Essity brand



Hydrofera® Dressings manufactured by:  
Hydrofera, LLC, 340 Progress Drive  
Manchester, CT 06042, USA  
Tel: 1.800.537.1063 Fax: 1.800.853.4325  
Email: [hms.orders@essity.com](mailto:hms.orders@essity.com)

65145 R2 ©2024 HMS North America A24