

# Hydrofera Blue READY® Tube Site Dressing

WOUND DRESSING  
APPLICATION GUIDE

## HydroferaBLUE®

The Hydrofera Blue READY® Tube Site Dressing is a polyurethane foam designed with an intuitive shape and specialized key-hole opening for tracheostomy tubes, drains and other tube sites to ensure precise placement and ease of application.



### Step 1: Prepare



Prepare the tube entry site by thoroughly cleansing the area, per your facility's protocol. Carefully remove any dried secretions or debris from the area.

### Step 2: Apply



- Gently open the sterile Hydrofera Blue READY® Tube Site Dressing package.



- Apply dressing to fit securely around the drain tube, with the writing on the film side facing away from the patient's skin.
- If fixation is needed, tape and secure it down. The tape should only be applied to the edges of the dressing.
- No secondary dressing is needed. Be careful not to apply too much pressure.

### Step 3: Monitor



Regularly monitor the site. If clinical conditions allow, the Hydrofera Blue READY® Tube Site Dressing can be left in place for up to 7 days or changed as advised by the medical professional.

- If the dressing is saturated more than 50%, change before 7 days.
- If the dressing turns white, replace with a new dressing.

**For more information, see Instructions for Use included with the product.**

Hydrofera®, Hydrofera Blue®, Hydrofera Blue READY®, Hydrofera Blue READY-Border®, Hydrofera Blue TRANSFER® and Hydrofera Blue ComfortCel® are registered trademarks of Hydrofera, LLC. ©2024



Hydrofera®,  
an Essity brand



Hydrofera® Dressings manufactured by:  
Hydrofera, LLC, 340 Progress Drive  
Manchester, CT 06042, USA  
To order: 1-860-337-7730  
Orders.HF@essity.com  
www.hydrofera.com

65360 RN ©2024 HMS North America K24