

Hydrofera Blue READY® foam.

No secondary dressing required.

Application

Step 1

- Cleanse the wound in accordance with established procedures
- Carefully dry the periwound skin
- Select and apply the appropriate dressing size to ensure the dressing will cover the entire wound
- No hydration is required



Place with printed film side up

Step 2

- Secure the dressing using a bandage or tape



Step 3

- Dressing can be left in place up to 7 days
- Can be used in conjunction with compression wraps and total contact casting



Hydrofera Blue TRANSFER® foam.

Secondary dressing required.

Application

Step 1

- Cleanse the wound in accordance with established procedures
- Carefully dry the periwound skin
- Select and apply the appropriate dressing size to ensure the dressing will cover the entire wound
- No hydration is required



Step 2

- Apply an absorptive secondary dressing (e.g. Gauze, Bordered Foam)



Step 3

- Dressing can be left in place up to 7 days
- Can be used in conjunction with compression wraps and total contact casting



Hydrofera Blue READY® foam Ordering Information

Stock No	Product Size	Quantity/Box	HCPC Codes*
HBRS2520	2.5" x 2.5" (6.4 cm x 6.4 cm)	10	A6209
HBRS4520	4" x 5" (10.2 cm x 12.7 cm)	10	A6210
HBRS8820	8" x 8" (20.3 cm x 20.3 cm)	10	A6211

Hydrofera Blue TRANSFER® foam Ordering Information

Stock No	Product Size	Quantity/Box	HCPC Codes*
HBRT2525	2.5" x 2.5" (6.4 cm x 6.4 cm)	10	A6209
HBRT4050	4" x 5" (10.2 cm x 12.7 cm)	10	A6210
HBRT8080	8" x 8" (20.3 cm x 20.3 cm)	10	A6211

The BSN Medical / Essity Inc. Reimbursement Disclaimer:

The Centers for Medicare & Medicaid Services (CMS) uses Palmetto GBA LLC., as the Pricing, Data Analysis, and Coding (PDAC) Contractor. The PDAC Contractor maintains the Product Classification List, located at <https://www4.palmettogba.com/pdac/dmecs/>. The Product Classification List is a searchable database containing products that have received HCPCS coding verification from PDAC and their corresponding HCPCS codes. BSN Medical Inc., however, makes no representations as to the accuracy of the information contained within the PDAC database, nor any representations as to whether its products are reimbursable under any government sponsored healthcare program and/or private insurance program. It is the provider's sole responsibility, in consultation with the insurer, if necessary, to determine medical necessity, ensure coverage criteria is met, and submit appropriate codes, modifiers, and charges for services rendered. RV: 12/24